GAMMA EMOTECA E CONGELATORI PER PLASMA

blood bank range
and plasma freezers

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medical devices

Blood and blood-derived products storage is from the time being subject to strict regulations. The accurate control and maintaining of the working temperature and the presence of an alarm system complete with battery back-up, are some of the main requirements to satisfy for their safety storage. The sophisticate instrumentation for the temperature control supplied with all the range grants the optimal storage conditions required and the sure reliability of Fiocchetti's blood banks and plasma-freezer, in compliance with all national and international regulatory requirements.

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*Fiocchetti's blood banks (**Emoteca** range) and **plasma-freezers** have been developed for the storage of blood and blood related products, in full compliance with 93/42/CE - class IIa regulation, which identifies these equipment as medical devices, and with DIN 58371.*

Emoteca range and plasma freezer series are offered with several storage capacities, internal fitting and different versions.

The standard equipment, beside ECT-F Touch controller, include:

- + *the DMLP Digital Monitor Touch module, for the recording of temperatures by means of an independent PT100 probe and back-up batteries*
- + *external terminal board, including two dry contacts to remote alarms, RS485 and RJ45 Ethernet ports*
- + *electric digital key lock with password controlled access list*
- + *product temperature simulation probe*
- + *drawers with dividers or shelves (according to the supplied model)*
- + *feet or castors to allow the cleaning of the floor*

Fiocchetti's Plasma freezers and blood banks (Emoteca range) are registered into the Italian national register of medical devices.

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- + **Plexiglas front** for drawers
- + **Wire shelf** in a-toxic plastic coated steel, supported by anti-tilt clips.
- + **Aluminium drawer**, totally extractible
- + **Stainless steel basket** mounted on anti-tilt side slides.
- + **Stainless steel rack**, dimensions cm.14x50x15H
- + **Castors kit**, nr.2 with brakes
- + **Voltage stabilizer**
- + **PT100 probe** (free contacts)
- + **Pass-through port with cap**, diam. mm 35
- + **Pass-through port** mm 15
- + **Internal socket** 220V
- + **Double internal socket** 220V with ON/OFF switch
- + **Temperatures certificate:** test report, minimum nr.10 probes for 12 hours
- + **Frequency** 60 Hz
- + **Voltage** 115V
- + **Wooden crate package**
- + **Wi-Fi Module:** plug-in accessory for monitoring, connection and control at the distance of the equipment
- + **Gsm Communicator:** plug-in module for the notification, via SMS, of the functioning and possible faulty condition of the equipment. Every ECT-F Touch controller can be equipped with a GSM module, thus becoming an independent unit sending and receiving different sort of SMS. Monitoring SMS: calling the SIM card of the refrigerator will generate an SMS containing all functional data. ALERT SMS: failures of faulty conditions are notified to the pre-set numbers. When equipped with back-up batteries, in case of power failure, the GSM will inform of this condition and of power restore, once this will be the case. COMMAND SMS: messages can be sent to modify the controller parameters, the turning on or off of the unit, to change the set-point.

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B+B= Lamiera plastificata bianca antibatterica interno ed esterno / Pre-coated antibacterial sheet steel, white colour, both inside and outside

B+X= Lamiera plastificata bianca antibatterica esterno e acciaio inox AISI 304 interno / Pre-coated antibacterial sheet steel, white colour, outside and AISI 304 stainless steel inside